

Cleaning Telemedicine Tablets

It is important to disinfect tablet between patients.
Please use the following recommendations:

- Clean case, keyboard, and back with purple top Sani-Wipes
- Clean screen with *PDI Easy Screen* alcohol pad
- Avoid getting moisture in any opening
- Don't submerge your Apple product in any cleaning agents.
- Avoid excessive wiping, which might cause damage.
- Unplug all external power sources, devices, and cables when cleaning.
- Don't use aerosol sprays, bleaches, or abrasives.

